



Treatment Protocols  
urogold100<sup>®</sup>  
MTS Spark Wave<sup>®</sup> Therapy



Sexual Medicine



Spark Wave  
Technology



# TREATMENT PROTOCOLS


## SEXUAL MEDICINE

### Approved Standard Indications

Indication	Energy Range (mJ/mm <sup>2</sup> )	Frequency (Hz)	Impulses / Session	Number of Sessions	Treatment Interval
Peyronie`s Disease (PD) / IPP (Induratio Penis Plastica)	0.12 - 0.17	2 - 4	900 - 1500	~3 - 6	1 - 2 weeks

*Recommended applicator:* OE050 / OE050S.


It is recommended to apply shock waves onto each affected area of the penis shaft where the plaques are located.

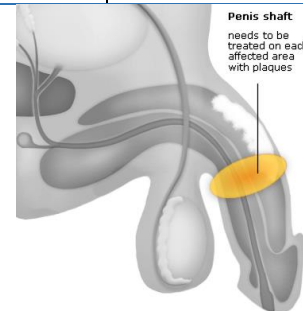

Erectile Dysfunction (ED)	0.10 - 0.14	2 - 4	1500 - 2500	~4 - 6	1 - 2 weeks
---------------------------	-------------	-------	-------------	--------	-------------

*Recommended applicator:* OP155 / OP155S.


It is recommended to apply the shock waves to the manually stretched penis to the distal, mid and proximal penile shaft and to the left and right crura of the penile shaft.

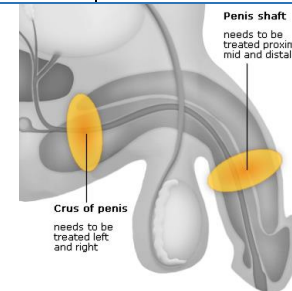

Chronic Pelvic Pain Syndrome (CPPS) / Chronic Prostatitis (CP)	0.12 - 0.14	2 - 4	900 - 1500	~3 - 6	1 - 2 weeks
--	-------------	-------	------------	--------	-------------

*Recommended applicator:* OP155 / OP155S or OE035 / OE035S.

Prior to treatment, prostate cancer must be excluded. It is recommended to apply the shock waves directly to the perineal area (from anus to scrotum) directed to where the pain has been localized.

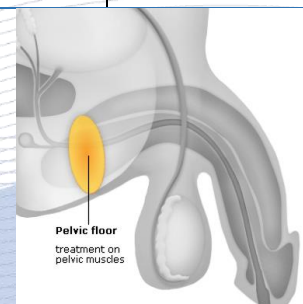

## Exceptional Indications


Indication	Energy Range (mJ/mm <sup>2</sup> )	Frequency (Hz)	Impulses / Session	Number of Sessions	Treatment Interval
Incontinence	0.12 - 0.14	2 - 4	1500 - 2500	~3 - 6	1 - 2 weeks

*Recommended applicator:* OP155 / OP155S or OE035 / OE035S.

Middle of the labia minora: ~800 pulses

Right side of labia minora: ~800 pulses


Left side of labia minora: ~800 pulses

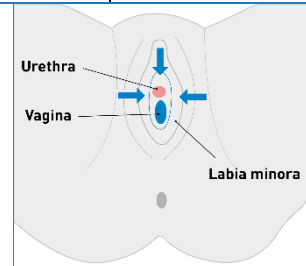

Interstitial Cystis / Bladder Pain Syndrome	0.12 - 0.14	2 - 4	900 - 1500	~3 - 6	1 - 2 weeks
---	-------------	-------	------------	--------	-------------

*Recommended applicator:* OP155 / OP155S or OE035 / OE035S.

Deliver SWs to the suprapubic region at five points: Points 1 and 2 at the level of transverse crease 2 cm above the pubic bone and 5 cm from each, points 3 and 4 are 2 cm above points 1 and 2, and point 5 is centered of points 1 – 4.

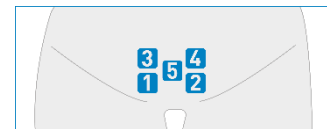

Premature Ejaculation	0.05 - 0.14	2 - 4	900 - 1500	~3 - 6	1 - 2 weeks
-----------------------	-------------	-------	------------	--------	-------------

*Recommended applicator:* OP155 / OP155S.


Men with PE are often more sensitive, so start the EFD low and respect the patient's tolerance level.

Distribute the SWs to the right and left lateral mid-shaft, the dorsal mid-shaft, the proximal ventral shaft and the distal ventral shaft overlying the frenulum.

Vulvodynia	0.05 - 0.14	2 - 4	3000	~3 - 6	1 - 2 weeks
------------	-------------	-------	------	--------	-------------

*Recommended applicator:* OP155 / OP155S.

Distribute the 3000 shocks: 1000 right / left lateral vestibule, and 1000 posterior vestibule.

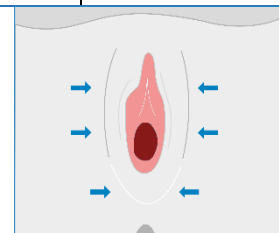

## Witness. The Spark of Life.