

Treatment Protocols

dermagold100[®]

MTS Spark Wave[®] Therapy



Dermatology



Aesthetics



Spark Wave
Technology



TREATMENT PROTOCOLS

DERMATOLOGY & AESTHETICS

Approved Standard Indications

Indication	Energy Range (mJ/mm ²)	Frequency (Hz)	Impulses / Session	Number of Sessions	Treatment Interval
Post Traumatic Lesions	0.08 - 0.13	2 - 4	350 + 20/cm ²	~3 - 6	1 - 2 weeks
Post-Operative Healing Disorders	0.08 - 0.13	2 - 4	350 + 20/cm ²	~3 - 6	1 - 2 weeks
Diabetic Foot Ulcer	0.08 - 0.13	2 - 4	350 + 20/cm ²	~3 - 6	1 - 2 weeks
Burns	0.08 - 0.13	2 - 4	350 + 20/cm ²	~3 - 6	1 - 2 weeks
Cellulite	0.08 - 0.13	2 - 4	~1500 / treatment site	~3 - 6	1 - 2 weeks
Scarring / Stretch marks	0.08 - 0.13	2 - 4	350 + 20/cm ²	~3 - 6	1 - 2 weeks
Lymphoedema	0.08 - 0.13	2 - 4	350 + 20/cm ²	~3 - 6	1 - 2 weeks

Recommended applicator: OP155 / OP155S.

Note: The number of shock waves results from the total surface of the affected area. The surface is calculated from the **total length x total width** of the affected area. (The wound periphery should also be treated: edge of the wound plus 1 cm).

Witness. The Spark of Life.



MTS Medical UG (limited liability)

Rev. 21-03-30

www.mts-medical.com